


# A Simple, SAFE Way of Removing Surgical Blades

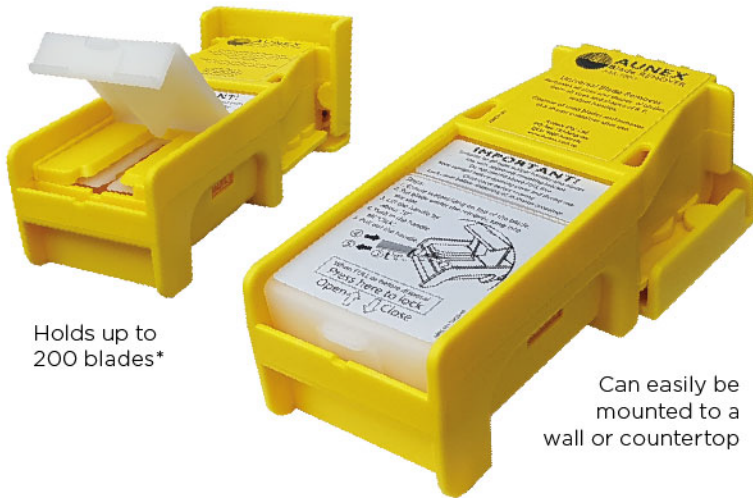

Holds up to 200 blades\*

Can easily be mounted to a wall or countertop

## Aunex Blade Remover

The only blade remover designed to remove *ALL* standard sized surgical blades, and is the first blade remover for size 60 and 70 blades.

Remove used blades with a single-handed technique. **No fingers, forceps or re-sheathing and NO INJURIES.**



## Instructions for Use:



### Step 1


To remove a blade, first lift the cover of the blade remover and insert the blade under the white rubber insert.

Ensure that the full handle extension is facing up and fully inserted. For most, this means the cutting edge of the blade will be facing to the right.




### Step 2

Next, slightly lift the back of the handle up about 10 degrees.

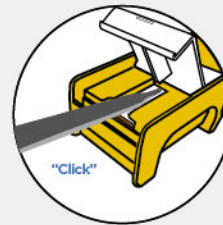

### Step 3

Then push in the handle until you hear a "click". This means the blade has been released.

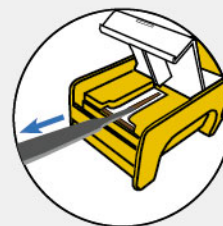

### Step 4

You are now safe to pull out the handle.

Check out our instructional video at [www.youtube.com/user/cincinnatiSurgical](http://www.youtube.com/user/cincinnatiSurgical)

[CincinnatiSurgical.com](http://CincinnatiSurgical.com)

The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this sheet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Cincinnati Surgical is excluded of all liability for injury to the extent permitted by law.

\*Depending on blade size